


Starting Guide

TIPS

When not in use, always charge LUNA® X5 by:

- Placing the hand control in the charging station.
- Connecting the charger cable to the side panel*.

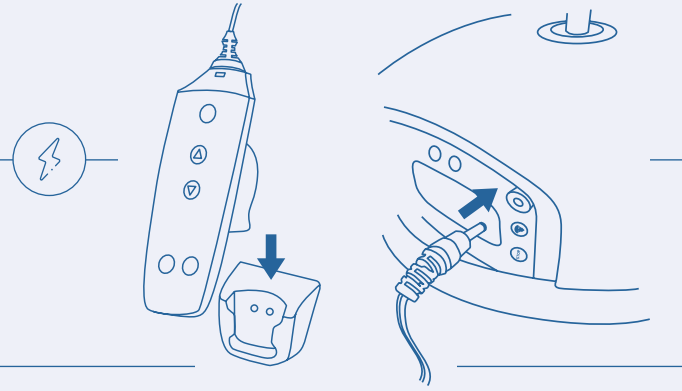

Do not pull the hand control cable to move the ceiling lift



Use the medical bed's adjustable height to make the transfer easier



Make sure you are using the right sling size for the resident

FREQUENTLY ASKED QUESTIONS

If the hand control does not work:

- ▶ Check that the emergency stop is not activated (if yes, use Reset button).
- ▶ Check that the hand control is properly connected.
- ▶ Check battery level.

Why is the charger light blinking?

The green light on the charger blinks when the lift is not charging. Please put the lift back to charge as described previously.



*If track system is equipped with ITC (In-Track Charging), charging occurs automatically.



SCAN FOR MORE INFO!

WINCARE



SWL 150 / 200 / 275 kg

WINCARE

Ceiling Lift System



Starting Guide


LUNA® X5

SHAPING THE FUTURE OF CARE

DESIGN . SIMPLICITY . MAXIMIZED VALUE . SUSTAINABILITY


Starting Guide

HAND CONTROL

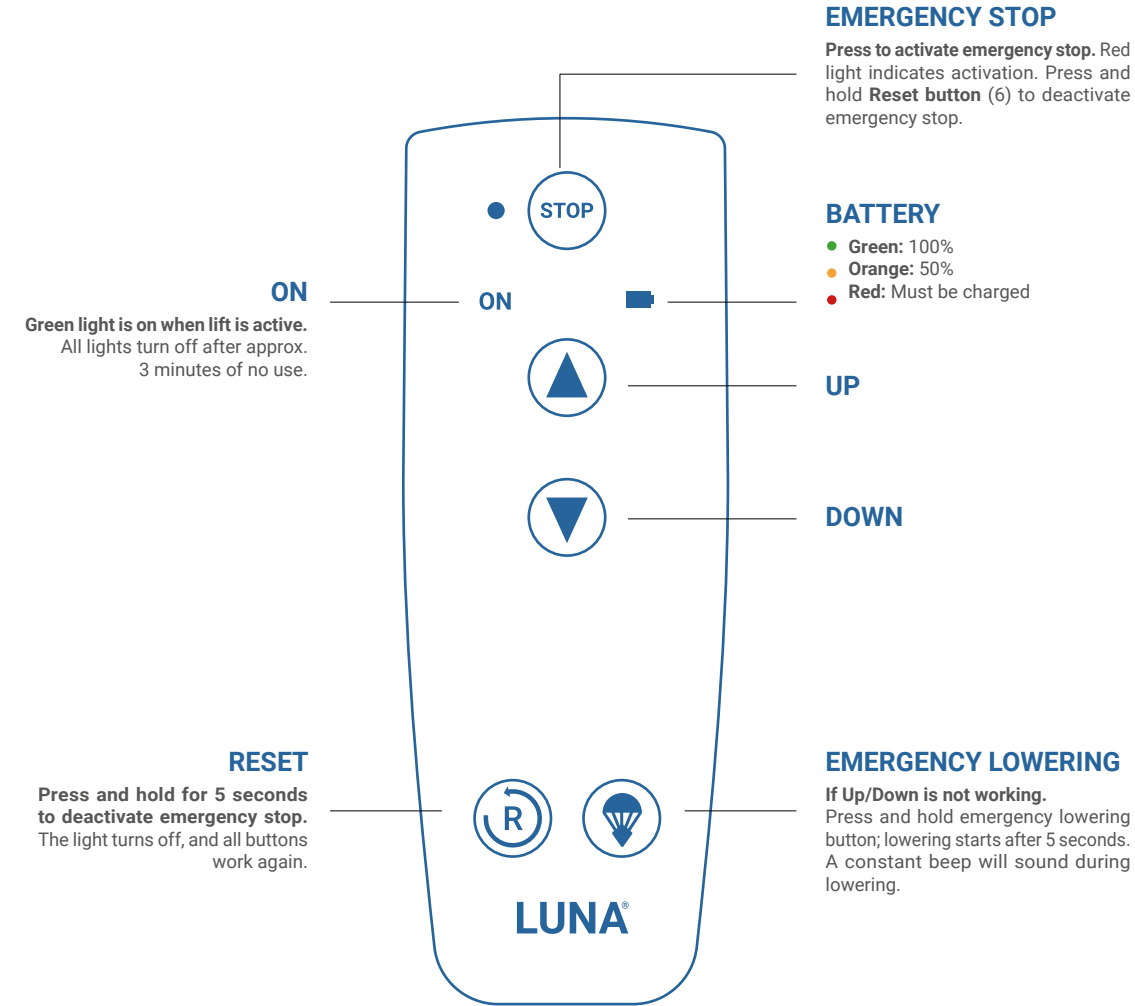

Starting Guide

SIDE PANEL

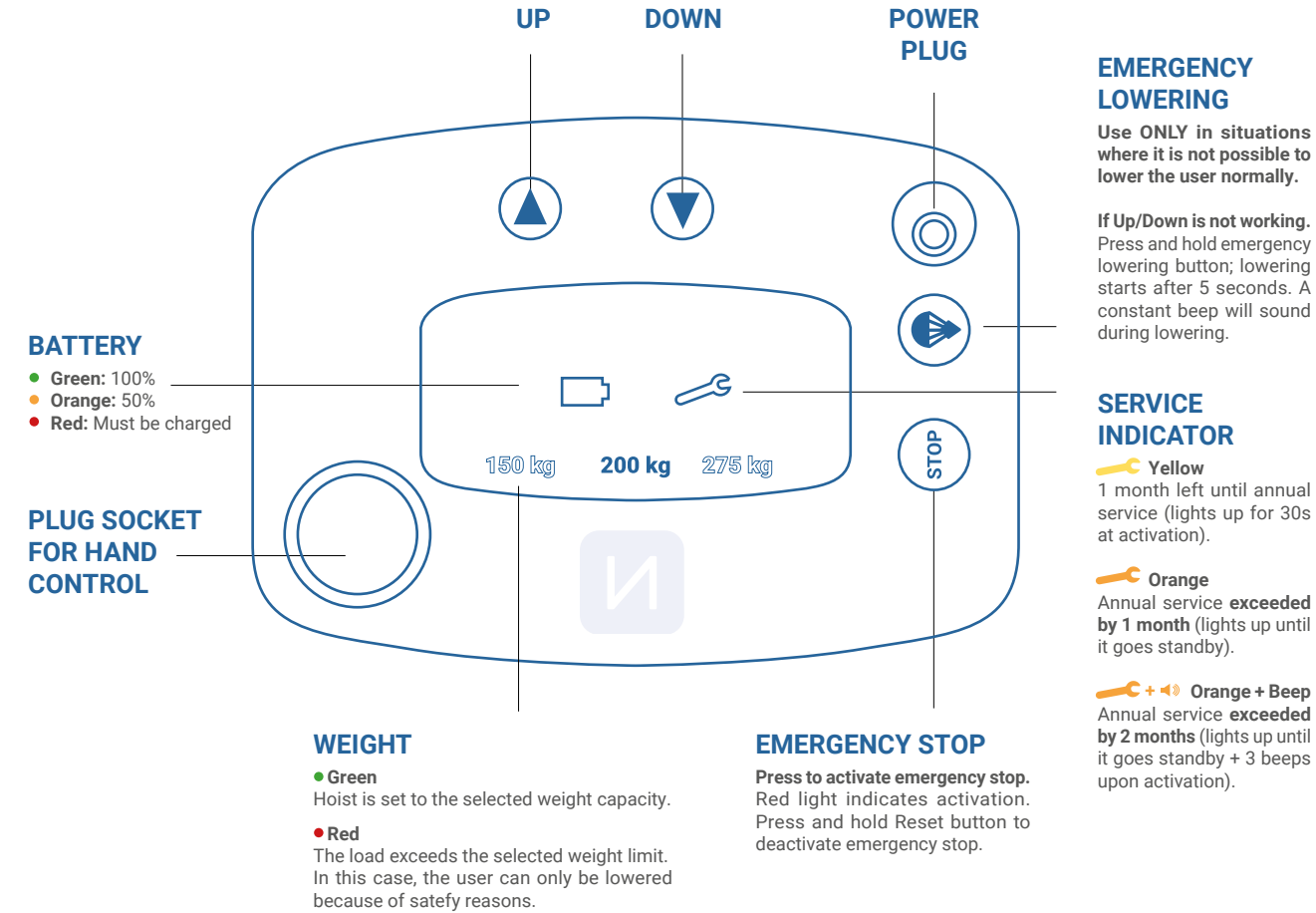

The display on the main lift panel automatically rotates when the lift is flipped over to ensure the indicators remain easy to read in any position.

How to move your lift FROM ONE ROOM TO ANOTHER

To switch your lift from fixed to portable mode... (or the other way around)

- In fixed mode, the lift is mounted directly in the track.
- In portable mode, the lift is mounted in the track with the strap.

...just turn the lift upside down.

