




SCAN FOR  
MORE INFO!

## Quick Guide

### HAND CONTROL

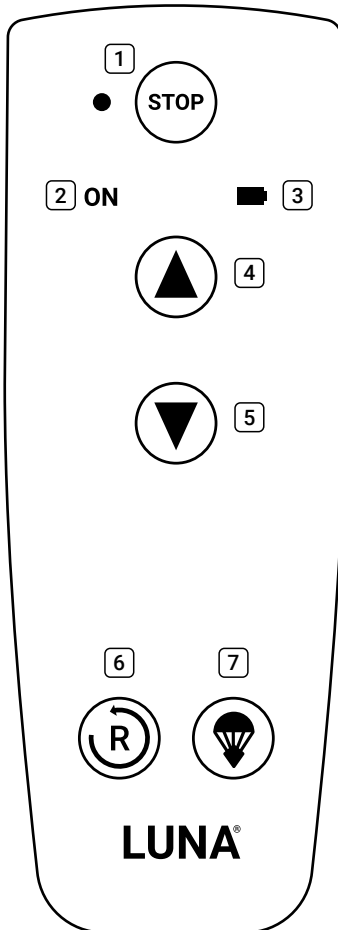

#### 1 Emergency stop

**Press to activate emergency stop.**  
Red light indicates activation. Press and hold Reset button (6) to deactivate emergency stop.

#### 2 ON

**Green light is on when lift is active.**  
All lights turn off after approx. 3 minutes of no use.

#### 3 BATTERY

- Green: 100%
- Orange: 50%
- Red: Must be charged

#### 4 UP

#### 5 DOWN

#### 6 RESET

**Press and hold for 5 seconds to deactivate emergency stop.** The light turns off, and all buttons work again.

#### 7 EMERGENCY LOWERING

**If Up/Down is not working.** Press and hold emergency lowering button; lowering starts after 5 seconds. A constant beep will sound during lowering.


#### Important

**When not in use, charge LUNA<sup>®</sup> X5 by:**

- Placing the hand control in the charging station.
- Connecting the charger cable to the side panel.

*If track system is equipped with ITC (In-Track Charging), charging occurs automatically.*

or

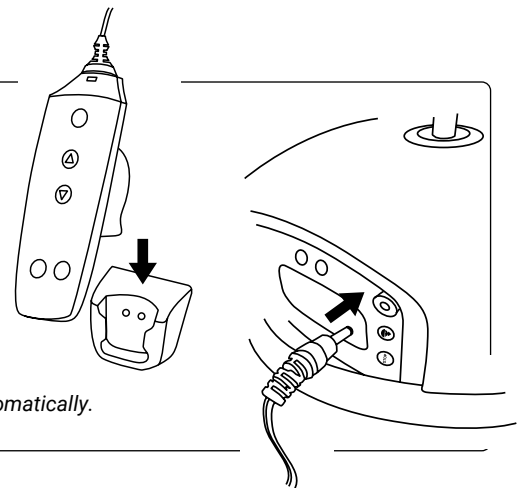

#### Emergency lowering

**If the electric emergency lowering is not working.**

Use the manual emergency lowering following these steps:

- a Pull out the emergency key located on the side of the LUNA<sup>®</sup> X5.
- b Insert the manual emergency key into the key hole.
- c Turn the manual emergency key counterclockwise for manual lowering.

