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Harness

Couraça de guindaste

Harnisches

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Imbracature per gru

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Manual and Warranty

Manual e Garanzia

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UK

Congratulations on your purchase of the Winncare Group harness crane.

The harnesses are made up of a wide range of products of different models and sizes, designed to cover the needs of any user, in accordance with their disability and on the type of transfer to be undertaken.

Our products comply with the requirements established in standard UNE-EN ISO 10535:2006: "Cranes for moving people with disabilities. Test methods and requirements".

Product according to the New Regulation (EU) 2017/745 "for Medical Devices.

#### **WARNINGS AND RECOMMENDATIONS:**

- The harness chosen must be of an appropriate size and suited to the user's disability.
- Do not use the harness if damage to the fabric, seams or materials are observed.
- Do not expose the harness to any heat source or allow it to come into contact with flames.
- Make sure that the straps are positioned correctly in the hanger ring.

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Arnés Confort 3D Arnés Confort 3D Arnés Confort 3D Arnés Confort 3D Arnés Confort 3D Arnés Confort 3D	Arnés Confort 3D piernas juntas Sangle Confort 3D ped joint Confort 3D harness legs together	Arnés Confort 3D apoyo cabeza Sangle Confort 3D tétière Confort 3D harness head support	Arnés Malla Comfort Harnais Confort Maille ComfortSling Mesh	Arnés Malla Comfort Harnais Confort Maille ped joint ComfortSling Mesh legs together	Arnés Malla Comfort Harnais Confort Maille tétière ComfortSling Mesh head support
102105012	102105014 *	102105016	102109012 102109012B	102109014 102109014B	102109016 102109016B
102105013	102105015 *	102105017	102109013	102109015	102109017

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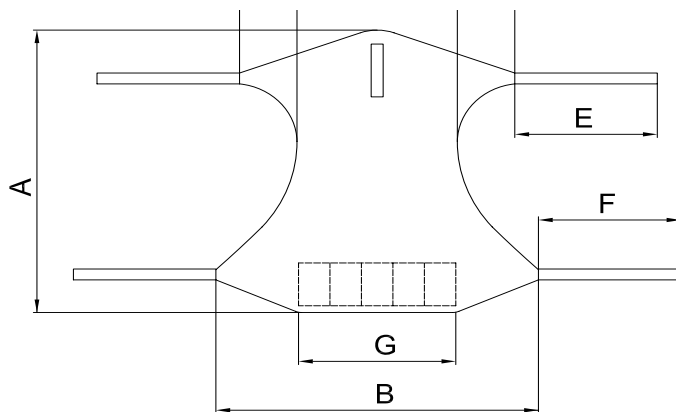
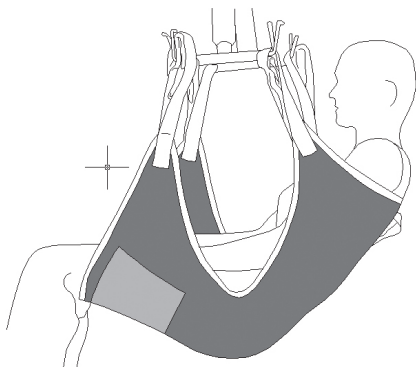
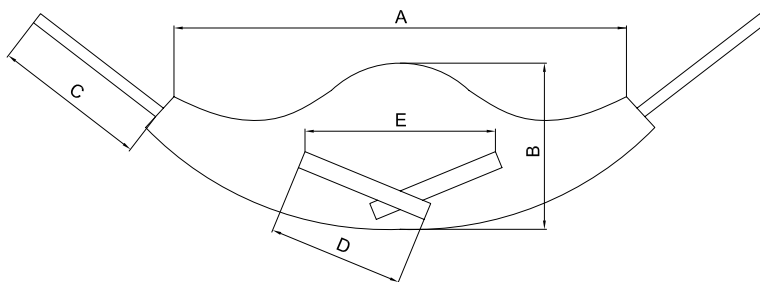
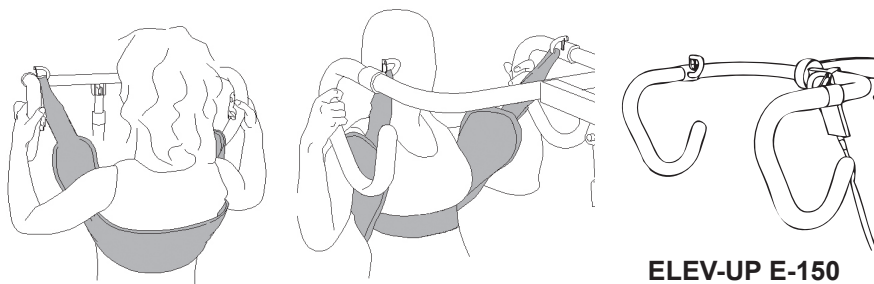
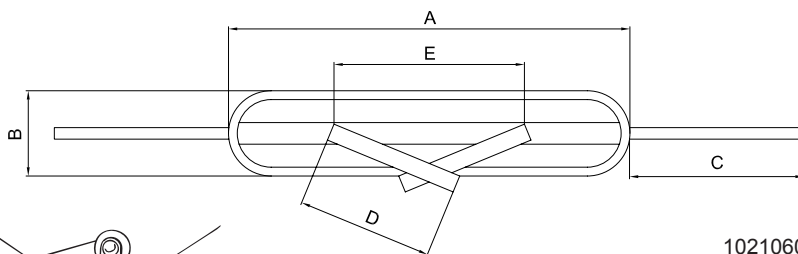
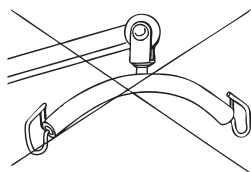
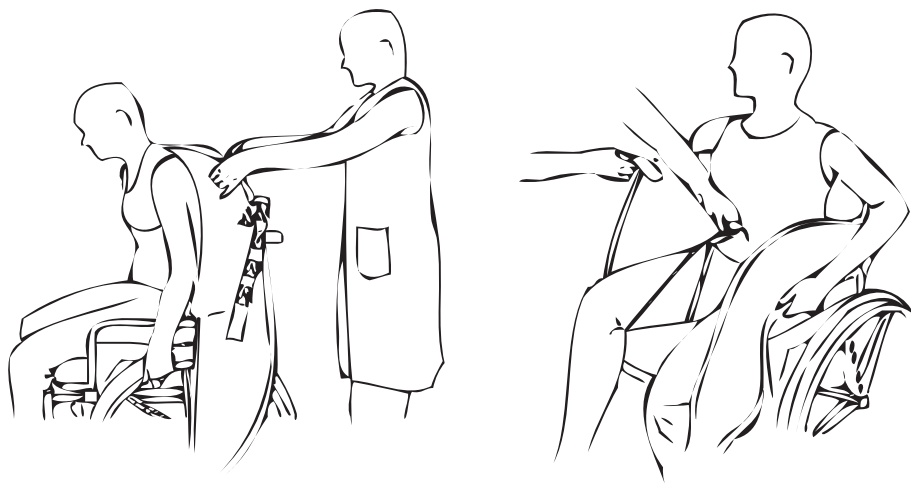
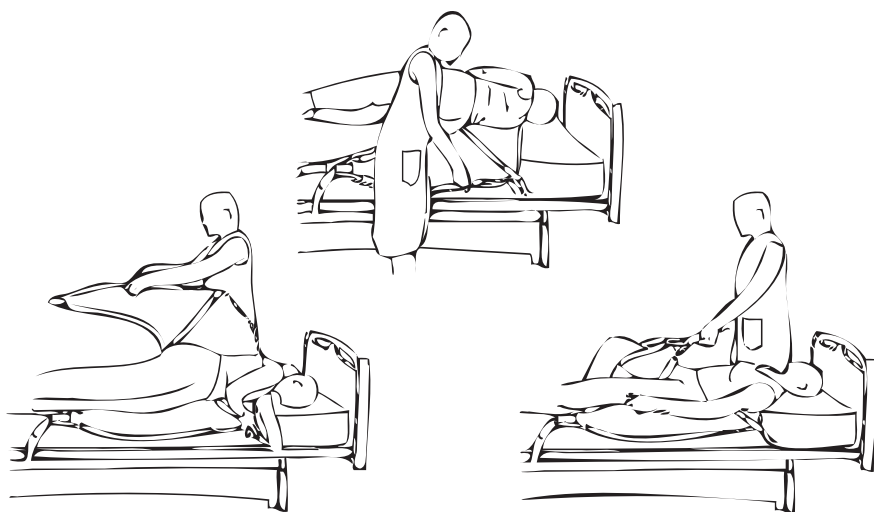
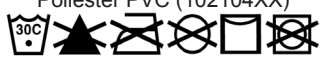
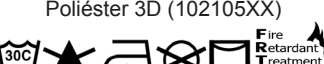
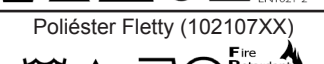
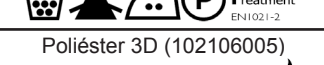
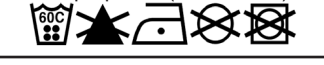
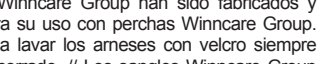
**UK** This Winncare Group product is guaranteed for 2 years and an estimated useful life of 5 years from the date of purchase under normal conditions of use, provided that the instructions are duly followed and that none of its components are changed or tampered with. If a defect in the product is detected, the distributor from whom it was purchased should be notified presenting him with the invoice or receipt, so that he may inspect it and, where appropriate and always with the manufacturer's agreement, decide on whether to repair or replace it providing that it is still within the guarantee period. According with Council Directive 85/374 / EEC of July 25, 1985, on the approximation of the laws, regulations and administrative provisions of the Member States regarding liability for damage caused by defective products, the distributor and / or WINNCARE will not be responsible for the damage caused by the defect that did not exist at the time they put the product into circulation or that this defect appeared later

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