

SLINGS FOR HOISTS

Compatible with Winn'Motion 150[®] and Winn'Motion 175[®]




HOIST SLINGS

MAIN SLING CHARACTERISTICS

- ▶ Spreader bar padded and secured by suspension points fitted with a non-return system.
- ▶ 2 sizes: M or L for patients weighing up to 175 kg.
- ▶ 3 settings for patients measuring 1.50 m to 1.90 m tall.
- ▶ With or without head support: for transferring patients with and without head support problem.
- ▶ Areas under the thighs are padded: for easier, more comfortable positioning of the patient.
- ▶ Visual cue to help with correctly positioning the sling: the yellow stripe indicates where the patient's spine should be aligned.
- ▶ Side handles to help reposition the patient in a chair. Back handle to help reposition the patient in bed.
- ▶ Compatible with 2- and 4-point spreader bars.

BATH

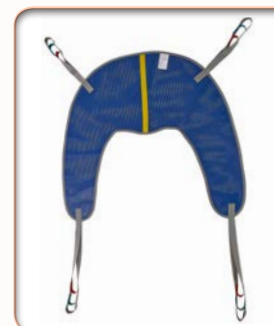

FLETTY SLING with head support



3D COMFORT



RECOMMENDATIONS FOR USE

TYPE OF TRANSFER	SLINGS	SLING BATH	SLING BATH SLING WITH HEAD SUPPORT	SLING FLETTY SLING	SLING FLETTY SLING WITH HEAD SUPPORT	SLING 3D COMFORT	SLING 3D COMFORT SLING WITH HEAD SUPPORT
Sitting ↔ Sitting		•	•	•	•	•	•
Sitting ↔ Lying		•	• (*)	•	• (*)	•	• (*)
Shower and Bath		•	•				
To and from the toilet from the ground		•	•				•
FUNCTIONAL STATE OF THE PATIENT	SLINGS	SLING BATH	SLING BATH SLING WITH HEAD SUPPORT	SLING FLETTY SLING	SLING FLETTY SLING WITH HEAD SUPPORT	SLING 3D COMFORT	SLING 3D COMFORT SLING WITH HEAD SUPPORT
Head instability			•		•		•
Contracture of the body						•	•
Amputation of the lower limbs			FOR AMPUTATION AT KNEE LEVEL: PASS EACH SLING UNDER BOTH THIGHS				

(*) Recommended

A COMPLETE RANGE OF SLINGS DESIGNED FOR USE WITH OUR HOISTS AND COMPLIANT WITH STANDARD EN 10535. (HOIST FOR TRANSFERRING DISABLED PATIENTS)

COMFORT

- ▶ the areas positioned under the patient's thighs are padded to enable better positioning of the sling. They thus guarantee that patients are held securely in place.

ERGONOMICS

- ▶ the yellow stripe indicating where the patient's spine should be aligned makes it easier to position the sling, particularly in lying position.
- ▶ The colored handles help to guide the patient during transfer operations.

3 MATERIALS FOR 3 DIFFERENT USES

- ▶ PVC for bath use.
- ▶ Fletty sling for routine transfers,
- ▶ 3D comfort for use with patients with specific disorders.

TRACEABILITY

- ▶ identification by batch number and year of manufacture.

	SLING BATH	SLING BATH SLING WITH HEAD SUPPORT	SLING FLETTY	SLING FLETTY SLING WITH HEAD SUPPORT	SLING 3D COMFORT	SLING 3D COMFORT SLING WITH HEAD SUPPORT
SIZE M	102 104 004	102 104 016	102 107 012	102 107 016	102 105 012	102 105 016
SIZE L	102 104 013	102 104 017	102 107 013	102 107 017	102 015 013	102 105 017
MATERIAL	POLYESTER PVC		POLYPROPYLENE		POLYESTER 3D	
QUALITY	WATERPROOF FABRIC		NON-ADHERENT FABRIC		BREATHABLE FABRIC	
WASHING AND CARE						
SPECIFIC FEATURES			PADDING UNDER THE THIGHS		PADDING UNDER THE THIGHS	
	FOR SLINGS WITH HEAD SUPPORT: REINFORCEMENT OF THE PART ON WHICH PATIENTS REST THEIR HEAD					

1-year warranty

Photos not contractually binding